



### HemCon® Dental Dressing 10mm x 12mm

#### Intended Use / Indications:

The HemCon® Dental Dressing is an oral wound dressing intended as a physical barrier for temporary protection of oral mucosal tissue and to provide pain relief.

#### Instructions:

1. Place the HemCon® Dental Dressing over the wound site.
2. Stabilize the HemCon® Dental Dressing by placing a cotton pressure dressing over the site and have the patient apply biting pressure.
3. Prior to discharging patient, visually inspect the site to confirm dressing placement. Replace cotton pressure dressing and dismiss patient.
4. The dressing usually dissolves within 48 hours, but may take up to 7 days.
5. At seven days irrigate (or have patient irrigate) the wound site to ensure removal of any residual material.

#### Cautions / Warnings:

Never suture HemCon® Dental Dressing in place using primary closure. An open wound is required when using this device. Do not use if seal is broken and/or contents are wet. Use promptly after opening. Contains chitosan from shellfish. Single use only. Do not resterilize.




#### Description:

The HemCon® Dental Dressing is a chitosan-based oral wound dressing intended as a physical barrier for temporary protection of oral mucosal tissue and to provide pain relief.

#### Storage Conditions:

Best stored at room temperature.

52° F (11° C)  78° F (26° C)

 **HemCon Medical Technologies, Inc.**, 10575 SW Cascade Ave., Suite 130, Portland, OR 97223 U.S.A.  
+1.503.245.0459 [www.hemcon.com](http://www.hemcon.com)

**Caution: U.S. Federal law restricts this device to sale by or on the order of a physician.**